

Hysteroscopic Diagnostic Trainer

- The Hysteroscopic Diagnostic Trainer **60326** is designed to meet the training needs of specialists and others working in the field of obstetrics and gynaecology.
- It is suitable both for doctors demonstrating the anatomy and pathology and for equipment manufacturers who are interested in promoting their hysteroscopes.
- The model provides an essential vehicle for repetitive practice with the hysteroscope and camera. All the uteri are all replaceable.

The Hysteroscopic Diagnostic Trainer comprises:

Universal Base	Re-order Code: 60325
Uterus Clamp	Re-order Code: 60334
Dilated Translucent Uteri	Re-order Code: 60316
Diagnostic Uterus with Polyps	Re-order Code: 60312
Diagnostic Uterus with Fibroid	Re-order Code: 60313
Diagnostic Uterus with Normal Anatomy	Re-order Code: 60322

Setting up a Diagnostic Uterus on the Universal Base

1. Release the uterus clamp by turning the clips on the flat surface of the base **fig 1**.

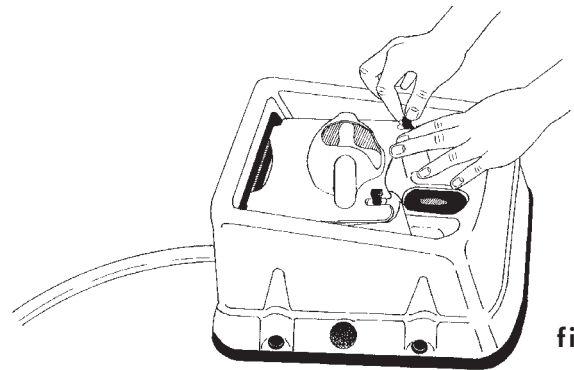


fig 1

2. Remove the clamp by lifting it out from the two slots on the back of the base **fig 2**.

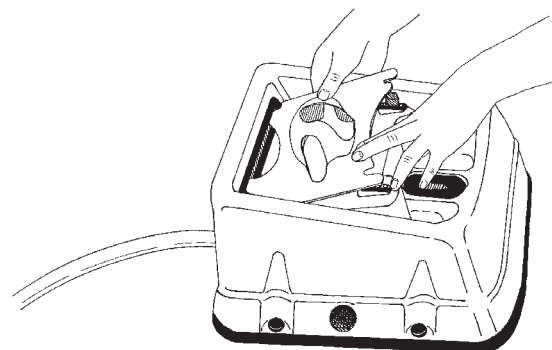


fig 2

3. A Diagnostic Uterus **fig 3**.

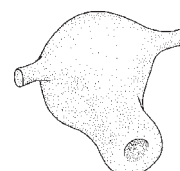


fig 3

4. Turn over the clamp and push the uterus into the hollow, easing the two extensions representing the fallopian tubes of the uterus through the holes on either side of the clamp **fig 4a**.

Turn over the clamp whilst supporting the uterus in position **fig 4b**.

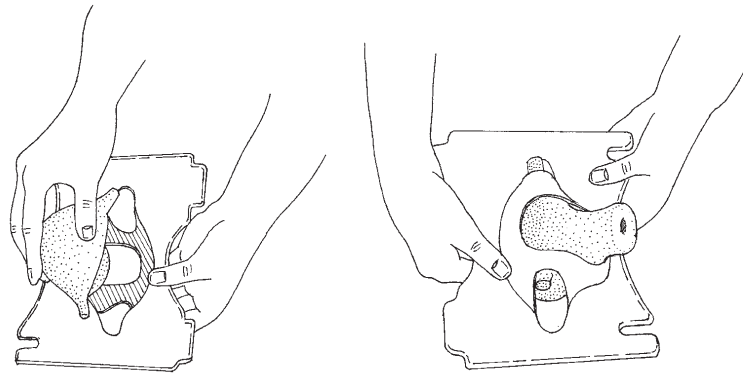


fig 4a

fig 4b

5. Slot the clamp with the uterus into place **fig 5**.

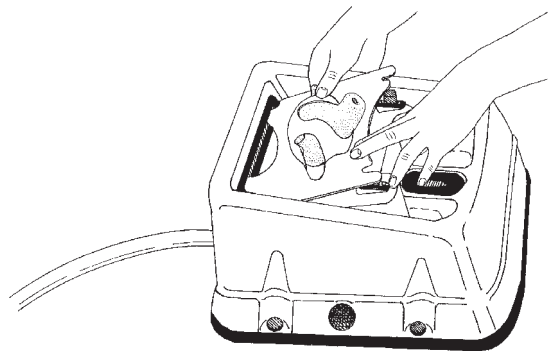


fig 5

6. Turn the clips to tighten **fig 6**.

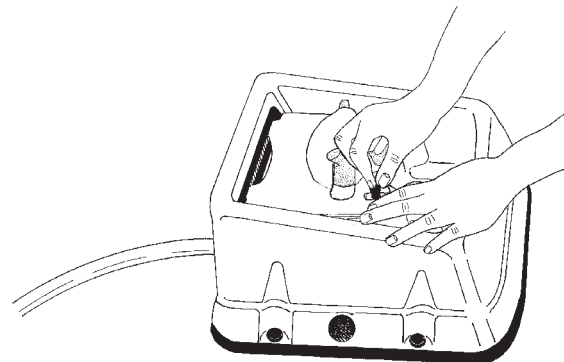


fig 6

7. The uterus is now in place **fig 7**.

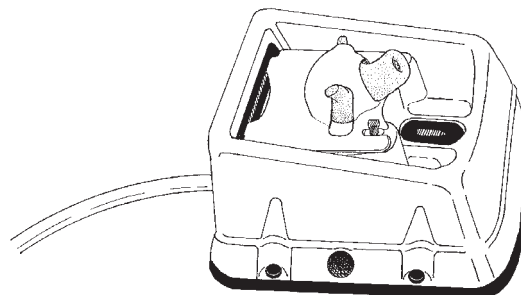


fig 7

Care of the product:

Please store in the airtight plastic bag provided.



Designed and manufactured by Limbs & Things Limited,
Sussex Street, St. Philips, Bristol, BS2 0RA, UK
Telephone: +44 (0)117 311 0500 Fax: +44 (0)117 311 0501
E-mail: sales@limbsandthings.com www.limbsandthings.com

L85-088
usehysd.qxd
Issue 3, December 2002
© Limbs & Things 2002